Dear Customer, This professional nebulizer is a high-quality device for inhalation therapy for the treatment and care of upper and medium/lower respiratory tract.


- 1. Important Safety Instructions
This device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect application.

NEB 10A
IB NEB 10A 10.01 1213

2. Preparation and Usage of this Device

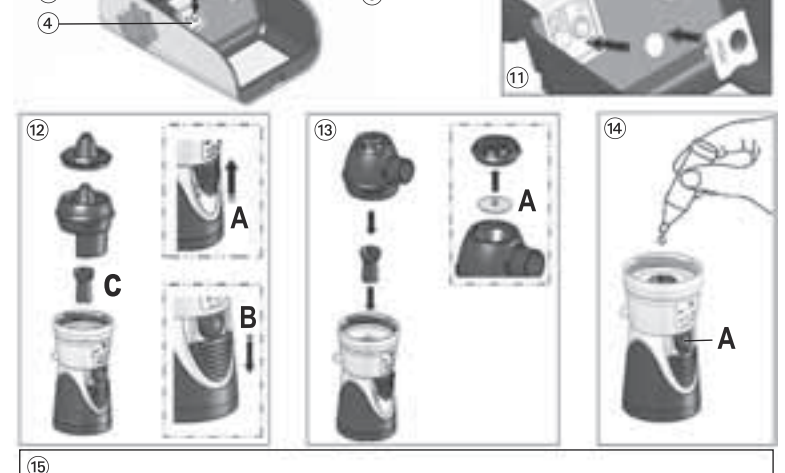- 1. Assemble the nasal washer (3) or the nebuliser kit (3).
2. Fill the nebuliser (4) with the inhalation solution as per your doctor's instructions. Ensure that you do not exceed the maximum level.

NEB 2 in 1 - Nasal washer
This device is equipped with the advanced «NEB 2 in 1» nasal washer for a complete, quick and effective treatment of upper airways (allergic and nonallergic rhinitis, rhinosinusitis, nasal polyps etc.).

4. NEB 2 in 1 - Aerosol therapy

The advanced «NEB 2 in 1» aerosol therapy allows effective treatments of the medium and lower respiratory tract (asthma, chronic bronchitis, cystic fibrosis etc.).

Short treatment
The therapeutic session will be reduced if the membrane is not mounted on the valve-holder.

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- 1. Piston compressor
2. Power lead
3. ON/OFF Switch
4. Air filter compartment
5. Nebuliser «NEB 2 in 1»

Dear Customer, This professional nebulizer is a high-quality device for inhalation therapy for the treatment and care of upper and medium/lower respiratory tract.

- 1. Important Safety Instructions
This device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect application.

NEB 2 in 1 - Nasal washer
This device is equipped with the advanced «NEB 2 in 1» nasal washer for a complete, quick and effective treatment of upper airways (allergic and nonallergic rhinitis, rhinosinusitis, nasal polyps etc.).

NEB 2 in 1 - Aerosol therapy
The advanced «NEB 2 in 1» aerosol therapy allows effective treatments of the medium and lower respiratory tract (asthma, chronic bronchitis, cystic fibrosis etc.).

4. NEB 2 in 1 - Aerosol therapy

The advanced «NEB 2 in 1» aerosol therapy allows effective treatments of the medium and lower respiratory tract (asthma, chronic bronchitis, cystic fibrosis etc.).

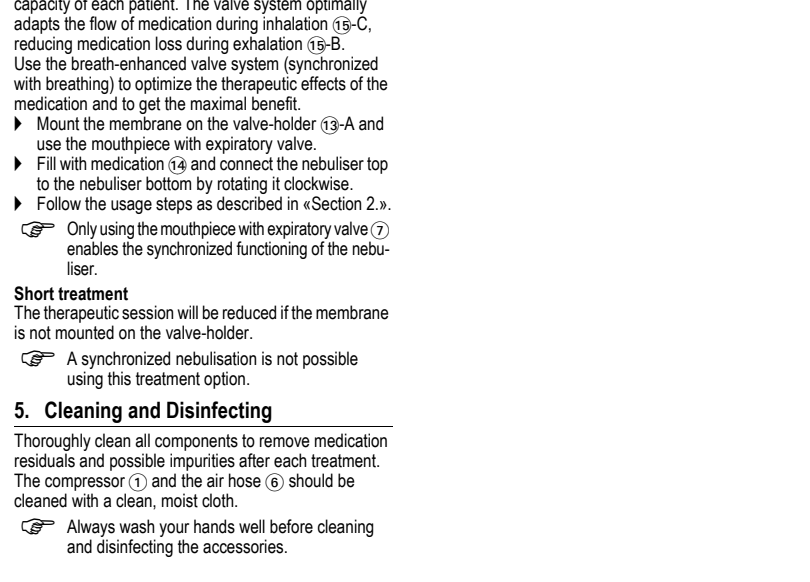
Short treatment
The therapeutic session will be reduced if the membrane is not mounted on the valve-holder.

Specialist Dealer / Специализированный дилер / Spezialist
This professional nebulizer is a high-quality device for inhalation therapy for the treatment and care of upper and medium/lower respiratory tract.

Cleaning and Disinfecting
Thoroughly clean all components to remove medication residues and possible impurities after each treatment.

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Professional Compressor Nebuliser EN

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3A Health Care S.r.l.